

CS4®



CS4® Spring



NEW SL Pivot
*Patent Pending



Class II



Class III

STEP 1:

Using our exclusive SL tool, lift the door to the open position.



STEP 2:

Place the SL pivot onto the archwire and the push door closed with index finger.



STEP 3:

Place Cs4® spring onto pivot and secure in place with hybrid screw.



**The SL Pivot should be used on wire sizes from .016×.025 up to .021×.025

IMPORTANT NOTE:
Hybrid screw must be "active" (full length) when used with SL pivot.

管理医療機器認証番号 229AKBZX00063000